

# Vinta-Zyme Plus

## Concentrated Enzymatic Detergent



### Product Description

Vinta-Zyme Plus is a low foaming enzymatic detergent designed to enhance the cleaning process in automated washers.

Vinta-Zyme Plus is designed for the automated cleaning of surgical instruments, medical devices, and other reusable items. It is a blend of five (5) enzymes that are effective in the removal of all protein, blood, fatty tissues, starches, etc. from all medical instruments, equipment, and supplies.

Specially formulated to solubilize and remove all bio-burden with a mixture of surfactants.

#### Applications

- Surgical instruments, endoscopes, medical devices
- Safe on: stainless steel, aluminum, anodized aluminum, titanium, brass, copper, common plastics

#### Advantages

- Concentrated, less storage and changeovers
- Multi-enzyme formula
- Works fast
- Surfactants remove bioburden efficiently

#### Instructions for Use

1. Add 1/4 to 3/4 oz per gallon (2 to 6 ml/L) of water
2. For best results, ensure water temperature is 80-110°F (26-44°C). Do not exceed 140°F (60°C)
3. Follow machine manufacturer's instructions and hospitals procedures

#### Technical Data

- pH neutral
- Bio-degradable, safe to use
- Non-toxic
- Contains corrosion inhibitors
- Low foaming
- Gentle on instruments

#### Common Packaging

- 4 – 1 Gallon Bottles
- 2 – 2.5 Gallon Pails
- 5 Gallon Pails
- 30 Gallon Drums
- 55 Gallon Drums

