

Vinta-Zyme Super

Super Concentrate Enzymatic Detergent



Product Description

Vinta-Zyme Super is a low foaming enzymatic detergent designed to enhance the cleaning process. Safe for use in all endoscope washers, washer disinfectors and ultrasonic cleaners.

Vinta-Zyme Super is designed for the automated cleaning of surgical instruments, medical devices, and other reusable items. It is a blend of five (5) enzymes that are effective in the removal of all protein, blood, fatty tissues, starches, etc. from all medical instruments, equipment, and supplies.

Specially formulated to solubilize and remove all bio-burden with a mixture of surfactants.

Applications

- Surgical instruments, endoscopes, medical devices
- Safe on: stainless steel, aluminum, anodized aluminum, titanium, brass, copper, common plastics

Advantages

- Super Concentrated, less storage and changeovers
- Multi-enzyme formula
- Works fast
- Surfactants remove bioburden efficiently

Instructions for Use

1. Add 1/8 to 3/8 oz per gallon (1 to 3 ml/L) of water
2. For best results, ensure water temperature is 80-110°F (26-44°C). Do not exceed 140°F (60°C)
3. Follow machine manufacturer's instructions and hospitals procedures

Technical Data

- pH neutral
- Bio-degradable, safe to use
- Non-toxic
- Contains corrosion inhibitors
- Low foaming
- Gentle on instruments

Common Packaging

- 4 – 1 Gallon Bottles
- 2 – 2.5 Gallon Pails
- 5 Gallon Pails
- 30 Gallon Drums
- 55 Gallon Drums